

### To remove the Permanent (fold-away) Mouthpiece

1. Start with the mouthpiece in the ready to test position (straight up & down)
2. Tilt the mouthpiece towards the closed position so that the mouthpiece is at a 45 degree angle (approximately). The mouthpiece should feel a bit loose and will have a slight wiggle towards the front and back of the LifeGuard instrument.
3. Using your thumb and index finger, hold the permanent mouthpiece just above the thumb tab and gently pull straight up and away from the LifeGuard instrument.



### To attach the Permanent (fold-away) Mouthpiece

1. To attach a new Permanent (fold-away) mouthpiece to the LifeGuard breath alcohol tester, hold the permanent mouthpiece between your thumb and index finger and line up the mouthpiece tabs (on either side of the mouthpiece) with the slots in the mouthpiece holder on top of the instrument.
2. Gently push the mouthpiece into the slots of the mouthpiece holder.
3. Tilt the mouthpiece to the test ready position to ensure the mouthpiece is securely fastened to the instrument.

